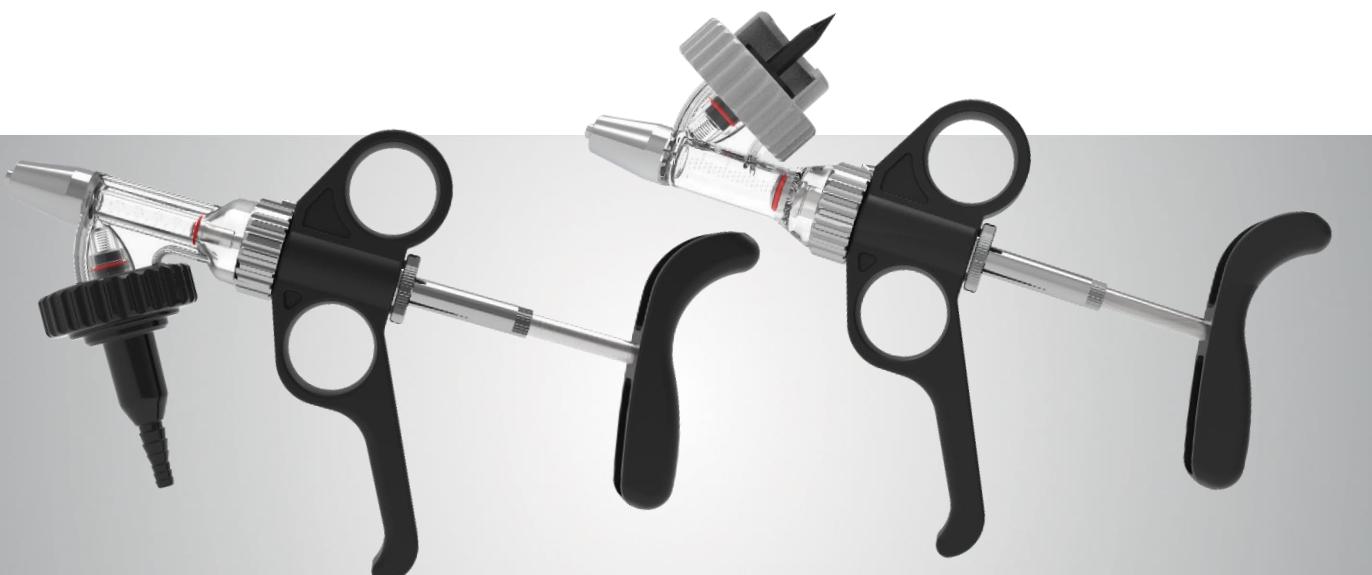
HSW B-MATIC 0,5ml, 1ml or 2ml

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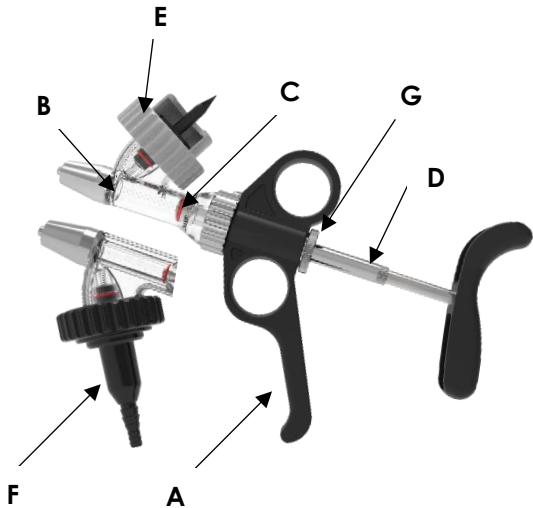
HSW B-MATIC 0,5ml, 1ml or 2ml

- Automatic self-filling syringe can be used for injection with bottle attachment or tube attachment

- A) Ergonomic handle
- B) Cylinder with bottle mount
- C) Piston with O-Ring
- D) Dosage setting
- E) Universaladapter
- F) Tubing Adapter
- G) Lock-nut

The B-MATIC is designed for robust use.

Thanks to the special design of the handle, the syringe can be easily converted from a bottle attachment to a tube attachment, depending on the version.

It is very easy to dismantle and clean.

The products must be cleaned and disinfected with suitable agents before each use.

Please follow the instructions of the manufacturer of the disinfectant you are using.

General warnings:

- ⚠ The B-MATIC is only intended to be used for the treatment of animals. The application on humans may cause damage to their health.
- ⚠ Do not point the B-MATIC towards any part of your body or towards other people! This can lead to injuries or accidental injections.
- ⚠ In case of accidental injection or application of a substance, consult a doctor immediately and present the information sheet of the respective substance to the doctor.
- ⚠ The use of non-approved substances or the overdosage / underdosage of these approved substances can lead to adverse health effects in animals and/or later in humans.
- ⚠ Caution, parts of the B-MATIC can break / be damaged. Damaged parts can cause injuries to the operator and animals.
- ⚠ Caution, sharp needles and broken needles can cause harm to the operator and animals. Use needles with care and do not bend them.
- ⚠ Pay attention to the spike in the vaccine vial holder area. It may contain residues of substances. In case of a caused spike wound, consult a doctor immediately and present the information sheet of the respective substance to the doctor.

General notes:

- Substances to be administered with the B-MATIC are not provided by HSW. The manufacturer's instructions must be adhered.
- The B-MATIC may only be used by operators who have read and understood the user manual in full.
- It is recommended to wear protective goggles, a mouth guard and rubber gloves during the use and cleaning of the B-MATIC.

Explanation packaging symbols

eIFU:

This symbol indicates that a digital instruction manual is available for the syringe. The digital instruction manual can be found under the following link: www.henkesasswolf.de/manuals/b-matic/ or by scanning the QR code shown on the right side.

Recycling codes:

The recycling codes provide information about the material used and can be helpful for waste separation.

The letter abbreviation PAP identifies cardboard and paper.
PAP 21 = Other cardboard

The letter abbreviation PE-LD identifies a particular plastic.
PE-LD 04 = Low density polyethylene

User Manual

The HSW B-MATIC 0.5ml, 1ml or 2ml was developed for injecting solutions. The ergonomic, user-friendly design and all high-strength materials used in its manufacture guarantee fatigue-free use and a long service life for your HSW B-MATIC.

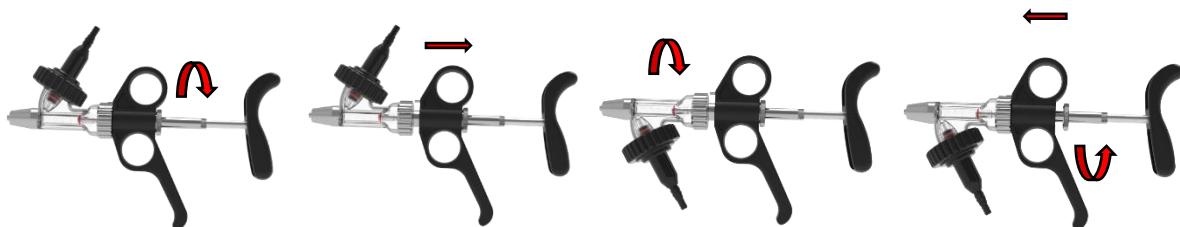
TO FIT THE NEEDLE

Attention:

- For your safety: Leave the protective cover (only on disposbale needles) on the needle until ready for use.
- Insert needle on to Luer Lock fitting and secure the needle clockwise.
- Disconnect the needle fitting the opposite way.

USING A TUBE FEED SYRINGE

- Turn the cylinder to the upper or lower position as required.

Open the lock nut so that you can push the handle backwards and the pin is free.

Turn the cylinder in the required position.

Push the handle forwards so that the pin secures the position and tighten the lock nut.

- Attach the tube to the tube adapter.
- Attach the tube to the bottle/injection solution.
- Before inserting the ascending tube into a new bottle - pierce the rubber stopper of the bottle with a sterile needle.

When using the syringe, observe the respective manufacturer's instructions for the medication!

CONNECT THE BOTTLE

- ⚠** Pierce the rubber cap of the bottle with sterile needle before you attach the bottle into the spike (If this is not observed, the vent valve can be pushed out of position and cause leaks).

Select the adapter for the corresponding bottle. Insert the bottle into the universal adapter

Insert the suction tip into the centre of the bottle.

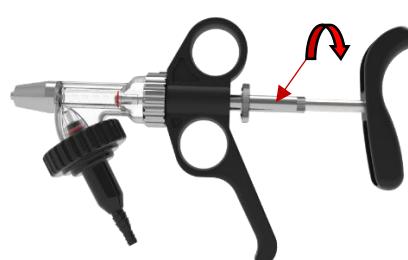
Tighten the adapter **without** turning the bottle


SET DOSE

Turn the Lock nut anti-clockwise and loosen.

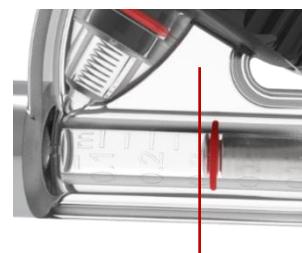Turn the dosage setting clockwise or anti-clockwise until the desired dose display on the cylinder is reached. The front edge of the piston serves as the dosing edge.
Tighten the lock nut

FILLING OF THE B-MATIC

- Remove portection cover of the needle.
- Hold the B-MATIC at a slight incline and take care that the spike is below the liquid to avoid sucking in of air into the barrel.
- Spueeze adjustable handle and then release quickly to draw fl uid into the barrel.
- To remove air bubble inside the barrel, repeat the step above
- When filling the syringe and preparing the injection, make sure that the syringe is set to 0.5 ml (for 0.5 ml syringes) and at least 0.5 ml (for 1 ml syringes) and that there are no air bubbles in the cylinder.

If necessary, point the cannula upwards so that the air can escape. After this initial setting, any dosing quantity can be set as required.

INSTRUCTION AFTER USE OF THE B-MATIC

Remove all traces of material from syringe by flushing thoroughly with a hot water detergent mix. Rinse thoroughly with clean, warm water.

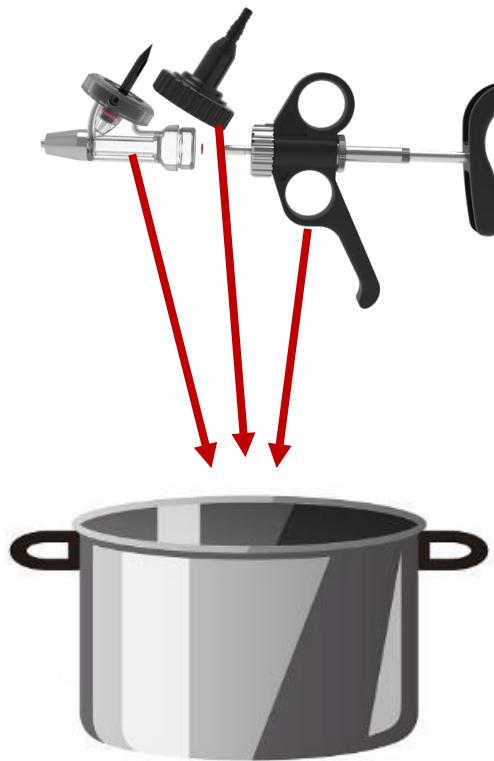
Caution!

If you notice that the syringe is drawing air after cleaning as described, it is possible that a valve has become stuck in the front metal attachment. The valve can be loosened by tapping the front part of the syringe lightly against an object, e.g. the edge of a table, and then rinsing the syringe with warm water.

DESINFECTION

It is strongly recommended to disinfect the B-MATIC and needles before each application.

- Clean the syringe with hot water. Use a bottle of warm water for this.
- Place the complete B-MATIC including inserted cannulas in a container of water and boil for approx. 20 minutes. Ideally, the cylinder should be removed from the handle and the syringe boiled in its disassembled state.
- Remove the B-MATIC after the boiling process, dry it with a cleaning cloth and pump it dry.

TO LUBRICATE

Lubricate the O-ring with medical white oil or alternatively 100% pure extra virgin olive oil, cold-pressed or equivalent edible oil. Observe the manufacturer's shelf life and storage instructions. Do not use mineral oils, silicone oils, WD-40, glycerine or alcohol. Petroleum-based oils can damage the piston or reduce the service life of the product. After oiling, the B-MATIC can be assembled (screw cylinder onto handle).

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© copyright · Henke-Sass, Wolf GmbH · 800020176- 07/24 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 800020176- 07/24 · Subject to amendments in technology and design · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 800020176- 07/24 · Reservadao los derechos de modificaciones técnicas y deseñico · Printed in Germany