




# **HOW TO USE EPIG®** SHORT USER GUIDE

This short user guide provides an overview of the most important steps for setting up EPIG® for usage and for cleaning the device after usage.

Prior to first usage of EPIG®, read the full user manual to avoid misuse of the device and danger to the user.


**1**

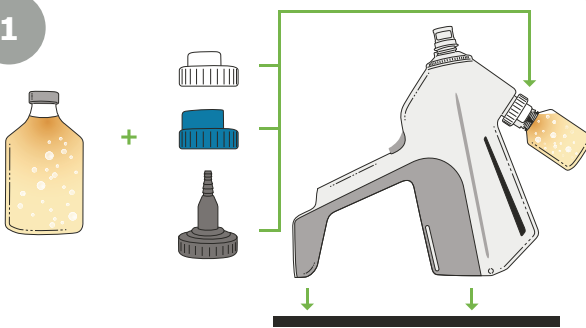

Take + Choose → Mount

Connect bottle with correct adapter, position the device on a solid surface and screw bottle onto spike.

**2**

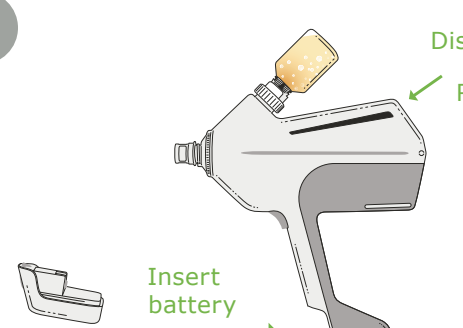


Insert battery

Display on = Ready

By inserting the battery, the device is switched on. The display will show the starting screen after 1. sec.


**3**

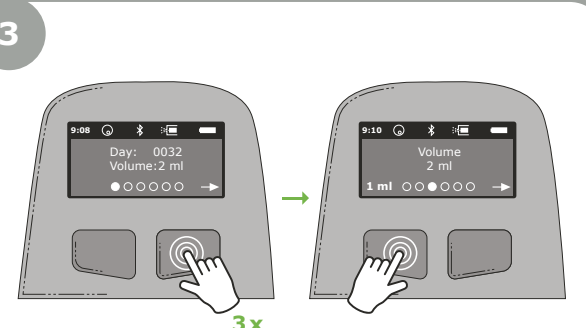

Startscreen 3x Change 1 ml/2 ml

Go through the user menu by pushing the right display button. Change a setting by using the left button.

**4**

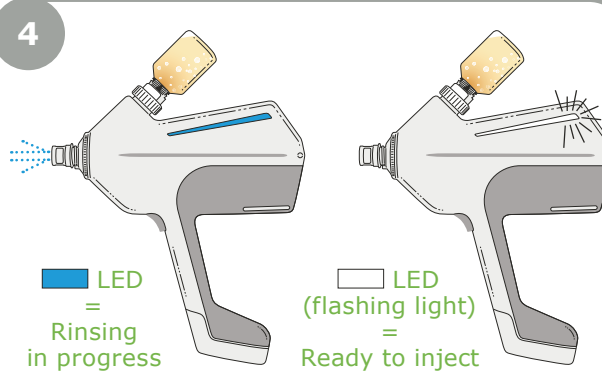


LED = Rinsing in progress

LED (flashing light) = Ready to inject

After setting the dose, push and hold trigger to prime the device (3 cycles) and getting ready to inject.

**5**

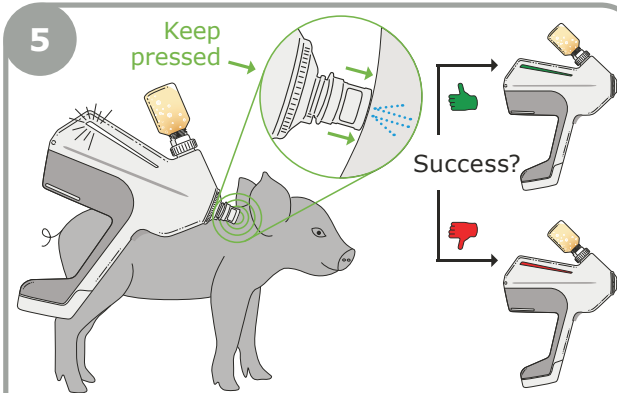


Keep pressed

Success?

Push and hold trigger and press the injection head against the neck of the piglet to release an injection.

**6**

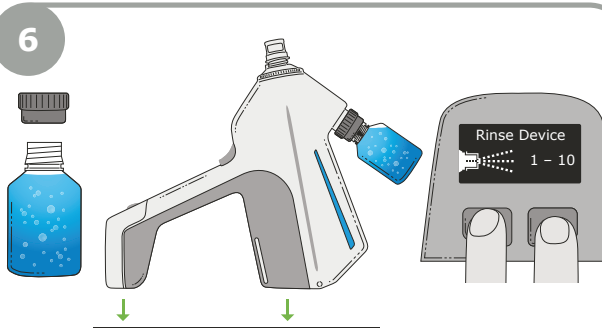

Take + Mount → Push 3 sec.

After usage, rinse device with specified cleaning fluid by pushing both display buttons for three seconds.