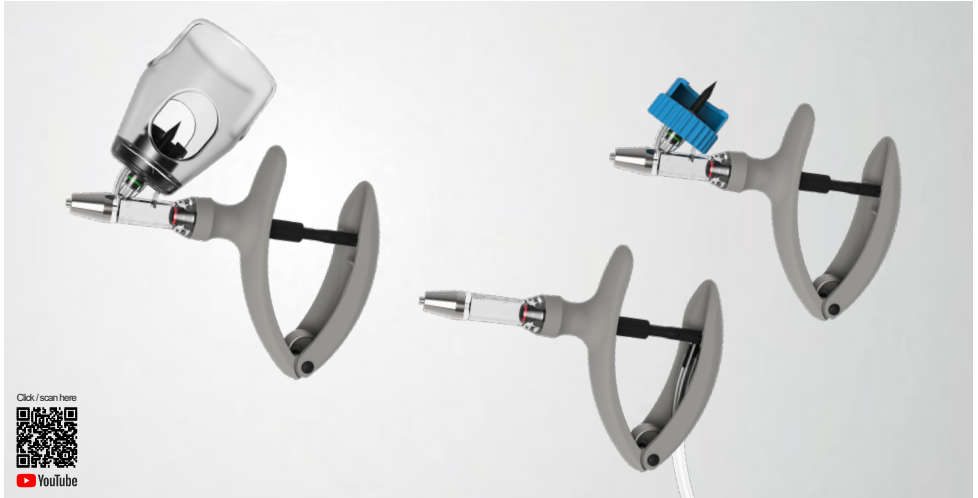
HSW ECO-MATIC®

- Bedienungsanleitung
- Instruction Manual
- Manual de instrucciones
- Mode d'emploi

For veterinary use only

CONTENT

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ENG

Automatic self-filling syringe for injection, with bottle attachment

- 1) Ergonomic designed handle
- 2) Plastic barrel with bottle attachment
- 3) Durable plastic basket
- 4) Metal needle fitting, Luer Lock
- 5) Dose setting
- 6) Grip hollow
- 7) Spike
- 8) View window

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Connect the bottle:

- Pierce the rubber cap of the bottle with a sterile needle before you attach the bottle into the spike.
- Insert the bottle into the basket.
- Line up the bottle, by using the viewing window of the basket, in the centre of the rubber cap and pierce the spike into the bottle while rotating the bottle slightly.

Exchange the bottle:

- Pull out the bottle by using the grip hollow on both sides of the bottle.

Set dose:

- For 0.5ml, 1 ml, 2ml and 5 ml: Turn the dose setting either clockwise or counter-clockwise until the front of the piston is aligned with the dose setting printed on the barrel.
- For 0,3 ml: Turn the dose setting either clockwise or counter-clockwise until the front of the O-ring is aligned with the dose setting printed on the barrel.
- In some syringe models the dosage is fixed at a certain volume and cannot be adjusted manually.

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Instructions after use:

- Remove all traces of material from syringe by flushing thoroughly with a hot water detergent mix. Rinse thoroughly with clean, warm water.
- ! *If you detect that the syringe draws air after the described cleaning, it is possible that a valve is fixed in the front metal approach. The valve can be released by knocking slightly the front of the syringe against an object, e.g., the edge of a table, and afterwards rinsing the syringe with warm water.*

Disinfection:

- ! *It is recommended that the HSW ECO-MATIC® and a supply of needles or nozzles be disinfected before / after each use.*
- Suck „hot“ water into the barrel by using an attached bottle with warm water.
- Suspend the complete HSW ECO-MATIC® including needles or nozzles in a container of water and boil it for up to 20 minutes. Ideally, the cylinder will be disassembled from the handle and the syringe will be boiled in a disassembled condition.
- Remove the HSW ECO-MATIC® from container, wrap cloth around the handle and pump it dry.

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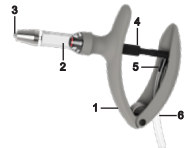
ENG

Automatic self-filling syringe for injection, with tube attachment

- 1) Ergonomic designed handle
- 2) Plastic barrel with piston and O-ring
- 3) Metal needle fitting - Luer Lock
- 4) Dosage adjustment
- 5) Tube attachment
- 6) Tube-hole in handle

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To fit the needle:

- ! *For your safety: Leave the protective cover (disposable needles) on the needle until ready for use.*
- Insert needle onto Luer Lock fitting and secure the needle clockwise.
- Disconnect the needle fitting the opposite way.

Attach draw tube into the inlet socket:

- Insert draw tube into the tube inlet socket of the syringe.
- Push the end of the draw tube through tube insert hole on the handle and straighten the tube.

Set dose:

- Turn the dose adjuster either clockwise or counter-clockwise until front of the piston is aligned with the dose setting printed on the barrel.
- In some syringe models the dosage is fixed at a certain volume and cannot be adjusted manually.

Priming:

- Remove protection cover on the needle.
- Hold syringe at a slight incline.
- Squeeze adjustable handle and then release quickly to draw fluid into barrel.
- To remove air bubbles inside barrel, repeat previous step taking care not to waste fluid.
- To fill the syringe and prepare for injection please make sure that the syringe is adjusted at least to 0.5 ml (for the 1 ml syringe) to prime and that no air bubbles are visible. Hold the syringe upright and squeeze the plunger so that air bubbles can be released. From now on, any dosage will be possible.

Instructions after use of the HSW ECO-MATIC®

- Remove all traces of material from syringe by flushing tho-

roughly with a hot water detergent mix. Rinse thoroughly with clean, warm water.

! If you detect that the syringe draws air after the described cleaning, it is possible that a valve is fixed in the front metal approach. The valve can be released by knocking slightly the front of the syringe against an object, e.g., the edge of a table, and afterwards rinsing the syringe with warm water.

Disinfection:

! It is recommended that the HSW ECO-MATIC® and a supply of needles or nozzles be disinfected before / after each use.

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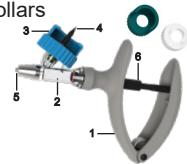
ENG

Automatic self-filling syringe for injection, with bottle mount universal collars

- 1) Ergonomic designed handle
- 2) Plastic barrel with bottle attachment
- 3) Universal collar
- 4) Spike
- 5) Metal needle fitting, Luer Lock
- 6) Dosage setting

The HSW ECO-MATIC® self-filling dosing syringe is designed for continuous use, only the O-ring and the barrel can be changed if necessary. It is essential that the syringe is cleaned before each use. Please follow recommendations specified herein with any cleaning or disinfectant solutions used.

To fit the needle:

- ! For your safety: Leave the protective cover (disposable needles) on the needle until ready for use.
- Insert needle onto Luer Lock fitting and secure the needle clockwise.
- Disconnect the needle fitting the opposite way.

Connect the bottle:

- Select the appropriate universal adapter for the bottle. White is for bottle neck diameters of 20 mm (50-100 ml), blue for 30 mm (100-250 ml), and green for 33 mm (>500 ml).
- Insert the bottle into the universal adapter.
- Insert the spike into the center of the bottle.
- Tighten the universal adapter without turning the bottle.
- *Optional accessories include a threaded bottle adapter for PET bottles or a tube adapter for connecting to the vaccine via a tube (Ø4-6 mm).*

Set dose:

- For 0.5ml, 1 ml, 2ml and 5 ml: Turn the dose setting either clockwise or counter-clockwise until the front of the piston is aligned with the dose setting printed on the barrel.

For 0,3 ml: Turn the dose setting either clockwise or counter-clockwise until the front of the O-ring is aligned with the dose setting printed on the barrel.

- In some syringe models the dosage is fixed at a certain volume and cannot be adjusted manually.

Priming:

- Remove protection cover of the needle.
- Hold the ECO-MATIC® at a slight incline and take care that the spike is below the liquid to avoid suck in of air into the barrel.
- Squeeze adjustable handle and then release quickly to draw fluid into the barrel.
- To remove air bubble inside the barrel, repeat step (3).
- To fill the syringe and prepare for injection please make sure that the syringe is adjusted on volume, 0,3 ml (for 0,3 ml syringe) 0,5 ml (for 0,5 ml syringe) and at least 0,5 ml (for 1ml syringe) to prime and that no air bubbles are visible. Hold the syringe upright and squeeze the plunger so that air bubbles can be released. From now on any dosage will be possible.

Instructions after use:

- Remove all traces of material from syringe by flushing thoroughly with a hot water detergent mix. Rinse thoroughly with clean, warm water.

! *If you detect that the syringe draws air after the described cleaning, it is possible that a valve is fixed in the front metal approach. The valve can be released by knocking slightly the front of the syringe against an object, e.g., the edge of a table, and afterwards rinsing the syringe with warm water.*

Disinfection:

! *It is recommended that the HSW ECO-MATIC® and a supply of needles or nozzles be disinfected before / after each use.*

- Suck „hot“ water into the barrel by using an attached bottle with warm water.
- Suspend the complete HSW ECO-MATIC® including needles or nozzles in a container of water and boil it for up to 20 minutes. Ideally, the cylinder will be disassembled from the handle and the syringe will be boiled in a disassembled condition.
- Remove the HSW ECO-MATIC® from container, wrap cloth around the handle and pump it dry.

Lubrication:

- Remove plastic barrel from the handle by turning it.
- Lubricate piston and glass barrel with 100% pure olive oil, cold squeezed or equivalent vegetable oil.
- Please follow the manufacturers durability and storage life instruction.
- Do not use mineral oil, silicone, WD-40, glycerine or alcohol.
- Petroleum based products harden the piston, reducing the life of the component.

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General warnings for all products of the HSW ECO-MATIC® family:

- ! The HSW ECO-MATIC® is only intended to be used for the treatment of animals. The application on humans may cause damage to their health.
- ! Do not point the HSW ECO-MATIC® towards any part of your body or towards other people! This can lead to injuries or accidental injections.
- ! In case of accidental injection or application of a substance, consult a doctor immediately and present the information sheet of the respective substance to the doctor.
- ! The use of non-approved substances or the overdosage / underdosage of these approved substances can lead to adverse health effects in animals and/or later in humans.
- ! Caution, parts of the ECO-MATIC® can break / be damaged. Damaged parts can cause injuries to the operator and animals.
- ! Caution, sharp needles and broken needles can cause harm to the operator and animals. Use needles with care and do not bend them.
- ! Pay attention to the spike in the vaccine vial holder area. It may contain residues of substances. In case of a caused spike wound, consult a doctor immediately and present the information sheet of the respective substance to the doctor.

General notes for all products of the HSW ECO-MATIC® family:

- Substances to be administered with the HSW ECO-MATIC® are not provided by HSW. The manufacturer's instructions must be adhered.
- The HSW ECO-MATIC® may only be used by operators who have read and understood the user manual in full.
- It is recommended to wear protective goggles, a mouth guard and rubber gloves during the use and cleaning of the HSW ECO-MATIC®.
- Transport and store in dry and ambient conditions.

Explanation of packaging symbols:

eIFU:

- This symbol indicates that a digital instruction manual is available for this syringe. The digital instruction manual can be found under the following link: www.henkesasswolf.de/manuals

Recycling codes:

- The recycling codes provide information about the material used and can be helpful for waste separation.
- The letter abbreviation PAP identifies cardboard and paper. PAP 21= Other cardboard
- The letter abbreviation PE-LD identifies a particular plastic. PE-LD 04 = Low density polyethylene

Disposal:

- The HSW ECO-MATIC® products can be disposed of in household waste.
- Due to the risk of injury, cannulas / needles must be disposed of as hazardous waste in accordance with the regulations of each country.

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Intended Use & Intended User:

Intended Use: Intramuscular / subcutaneous vaccination & intranasal application of livestock

Intended User: Vaccination as well as maintenance and repair can be carried out by a veterinarian or instructed / trained farm staff.

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You can download an updated version of the complete spare part list for this product under www.henkesasswolf.de or kindly request the same.

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© copyright Henke-Sass, Wolf GmbH