-  Bedienungsanleitung
-  Instruction Manual
-  Manual de instrucciones
-  Mode d'emploi

CONTENT

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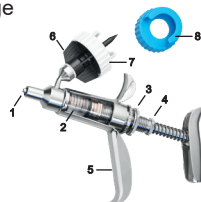
ENG

Automatic self-filling syringe

- 1) Metal needle hub, LL
- 2) Glass cylinder
- 3) Screw cap
- 4) Dosing adjustment piston
- 5) Ergonomic handle
- 6) Universal bottle holder
- 7) Bottle adapter, Ø20mm
- 8) Bottle adapter, Ø30mm

The HSW FERRO-MATIC® is designed for heavy-duty use. The syringe is easy to disassemble and clean. The product must be cleaned and disinfected with appropriate agents after each use. Please follow the instructions provided by the manufacturers of the disinfectants you use.

Inserting the needle:

- ! Please don't remove the protective cap from the needle (when using disposable needles) until you're ready to use the syringe.
- Attach the needle to the Luer Lock connector and secure it by turning the needle connector clockwise.
- To remove the needle, follow the steps in reverse. Replace the protective cap.

Attaching the bottle:

- Select the appropriate universal adapter for the bottle. White is for bottle necks with a diameter of 20 mm (50–100 ml), blue for 30 mm (100–250 ml).
- Insert the bottle into the universal adapter.
- Insert the suction tip into the center of the bottle.
- Tighten the universal adapter without turning the bottle.
- i Optional accessories include a threaded bottle adapter for PET bottles or a tubing adapter for connecting to the vaccine via tubing (Ø 4–6 mm).

Adjusting the dosing volume:

- Loosen the screw cap by turning it counterclockwise.
- Turn the dosing sleeve clockwise or counterclockwise until the desired dose is indicated on the cylinder. The front edge of the plunger serves as the dosing mark. Tighten the screw cap.

Preparing for use:

- Remove the protective cap from the cannula.
- Hold the FERRO-MATIC® at an angle and ensure that the suction tip is below the liquid level to prevent air from being drawn into the cylinder.
- Pump the syringe several times until the cylinder is filled with the desired dose and no air bubbles are visible.

Maintenance after use:

- After application, remove any residue using standard cleaning agents and rinse the syringe thoroughly under warm water.

Disinfection:

- ! It is strongly recommended to disinfect the product and needles before and after each use.
- Clean the syringe with hot water.

- Place the entire syringe, including the attached needles, in a container of water and boil it for a maximum of 20 minutes.
- After boiling, remove the syringe, dry it with a cleaning cloth, and flush it with water until dry.

Lubrication:

- Loosen the adjusting screw on the piston rod and remove the cylinder.
- Lubricate the piston with 100% pure extra virgin olive oil, cold-pressed, or an equivalent edible oil.
- Please follow the manufacturer's instructions regarding shelf life and storage.
- Do not use mineral oils, silicone oils, WD-40, glycerin, or alcohol.
- Petroleum-based oils can damage the O-rings or reduce the product's service life.

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General warnings for all products of the HSW FERRO-MATIC® family:

- ! The HSW FERRO-MATIC® is only intended to be used for the treatment of animals. The application on humans may cause damage to their health.
- ! Do not point the HSW FERRO-MATIC® towards any part of your body or towards other people! This can lead to injuries or accidental injections.
- ! In case of accidental injection or application of a substance, consult a doctor immediately and present the information sheet of the respective substance to the doctor.
- ! The use of non-approved substances or the overdosage / underdosage of these approved substances can lead to adverse health effects in animals and/or later in humans.
- ! Caution, parts of the FERRO-MATIC® can break / be damaged. Damaged parts can cause injuries to the operator and animals.
- ! Caution, sharp needles and broken needles can cause harm to the operator and animals. Use needles with care and do not bend them.
- ! Pay attention to the spike in the vaccine vial holder area. It may contain residues of substances. In case of a caused spike wound, consult a doctor immediately and present the information sheet of the respective substance to the doctor.

General notes for all products of the HSW FERRO-MATIC® family:

- Substances to be administered with the HSW FERRO-MATIC® are not provided by HSW. The manufacturer's instructions must be adhered to.
- The HSW FERRO-MATIC® may only be used by operators who have read and understood the user manual in full.
- It is recommended to wear protective goggles, a mouth guard and rubber gloves during the use and cleaning of the HSW FERRO-MATIC®.
- Transport and store in dry and ambient conditions.

Explanation of packaging symbols:

eIFU:

- This symbol indicates that a digital instruction manual is available for this this syringe. The digital instruction manual can be found under the following link: www.henkesasswolf.de/manuals

Recycling codes:

- The recycling codes provide information about the material used and can be helpful for waste separation.
- The letter abbreviation PAP identifies cardboard and paper. PAP 20 = Corrugated cardboard

Disposal:

- The HSW FERRO-MATIC® products can be disposed of in household waste.
- Due to the risk of injury, cannulas / needles must be disposed of as hazardous waste in accordance with the regulations of each country.

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Intended Use & Intended User:

Intended Use: Intramuscular / subcutaneous vaccination & intranasal application of livestock

Intended User: Vaccination as well as maintenance and repair can be carried out by a veterinarian or instructed / trained farm staff.

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Subject to amendments in technology and design

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You can download an updated version of the complete spare part list for this product under www.henkesasswolf.de or kindly request the same.

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© copyright Henke-Sass, Wolf GmbH