# HSW MULTI-MATIC®

-  Bedienungsanleitung
-  Instruction Manual
-  Manual de instrucciones
-  Mode d'emploi

For veterinary use only

## CONTENT

|                                                            |          |
|------------------------------------------------------------|----------|
| <b>HSW MULTI-MATIC®</b>                                    | <b>2</b> |
| GER                                                        | 2        |
| ENG                                                        | 2        |
| ESP                                                        | 3        |
| FRA                                                        | 3        |
| <b>General notes and warnings</b>                          | <b>5</b> |
| <b>Intended use, copy right and spare part information</b> | <b>8</b> |

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25ml, 50ml

### ENG

#### Dosing syringe for multiple dosing of livestock

- 1) Metal needle hub, LL
- 2) High-strength plastic cylinder
- 3) Piston with O-ring
- 4) Dosing wheel
- 5) Locking button
- 6) Ergonomic handle

The HSW MULTI-MATIC® is designed for heavy-duty use. The syringe is easy to disassemble and clean. The product must be cleaned and disinfected with appropriate agents after each use. Please follow the instructions provided by the manufacturers of the disinfectants you use.

#### Inserting the needle:

- ! Please don't remove the protective cap from the needle (when using disposable needles) until you're ready to use the syringe.
- Attach the needle to the Luer Lock connector and secure it by turning the needle connector clockwise.
- To remove the needle, follow the steps in reverse. Replace the protective cap.

#### Priming the syringe:

- Hold the syringe only by the front/fixed part of the handle.
- Press the lock button to pull the plunger rod backward.
- Pull the plunger rod backward and draw up the medication.
- Press the rear/movable part of the handle to lock the plunger rod back into place.
- The syringe is now ready for use.

#### Adjusting the dosing volume:

- The desired dosage can be set on the dosing wheel, depending on the model:
  - in 1-ml increments for the 50-ml model: 1-2-3-4-5 ml
  - in 0.5-ml increments for the 25-ml model: 0.5-1-1.5-2-2.5 ml

#### Maintenance after use:

- After application, remove any residue using standard cleaning agents and rinse the syringe thoroughly under warm water.

#### Disinfection:

- ! It is strongly recommended to disinfect the product and needles before and after each use.
- Clean the syringe with hot water. Use a bottle filled with hot water for this.
- Place the entire syringe, including the attached needles, in a container of water and boil it for a maximum of 20 minutes. (Optionally, the cylinder, including the metal cannula attachment, can be autoclaved at 134°C / 273°F.)
- After boiling, remove the syringe, dry it with a cleaning cloth, and flush it with water until dry.

#### Lubrication:

- Remove the barrel by turning it counterclockwise.
- Lubricate the plunger and O-ring with 100% pure extra virgin olive oil, cold-pressed, or an equivalent edible oil.
- Please follow the manufacturer's instructions regarding shelf life and storage.
- Do not use mineral oils, silicone oils, WD-40, glycerin, or alcohol.
- Petroleum-based oils can damage the O-rings or reduce the product's service life.

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## General warnings for all products of the HSW MULTI-MATIC® family:

- ! The HSW MULTI-MATIC® is only intended to be used for the treatment of animals. The application on humans may cause damage to their health.
- ! Do not point the HSW MULTI-MATIC® towards any part of your body or towards other people! This can lead to injuries or accidental injections.
- ! In case of accidental injection or application of a substance, consult a doctor immediately and present the information sheet of the respective substance to the doctor.
- ! The use of non-approved substances or the overdosage / underdosage of these approved substances can lead to adverse health effects in animals and/or later in humans.
- ! Caution, parts of the MULTI-MATIC® can break / be damaged. Damaged parts can cause injuries to the operator and animals.
- ! Caution, sharp needles and broken needles can cause harm to the operator and animals. Use needles with care and do not bend them.

## General notes for all products of the HSW MULTI-MATIC® family:

- Substances to be administered with the HSW MULTI-MATIC® are not provided by HSW. The manufacturer's instructions must be adhered to.
- The HSW MULTI-MATIC® may only be used by operators who have read and understood the user manual in full.
- It is recommended to wear protective goggles, a mouth guard and rubber gloves during the use and cleaning of the HSW MULTI-MATIC®.
- Transport and store in dry and ambient conditions.

## Explanation of packaging symbols:

eIFU:

- This symbol indicates that a digital instruction manual is available for this this syringe. The digital instruction manual can be found under the following link: [www.henkesasswolf.de/manuals](http://www.henkesasswolf.de/manuals)

Recycling codes:

- The recycling codes provide information about the material used and can be helpful for waste separation.
- The letter abbreviation PAP identifies cardboard and paper. PAP 20 = Corrugated cardboard

## Disposal:

- The HSW MULTI-MATIC® products can be disposed of in household waste.
- Due to the risk of injury, cannulas / needles must be disposed of as hazardous waste in accordance with the regulations of each country.

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## **Intended Use & Intended User:**

*Intended Use: Intramuscular / subcutaneous vaccination & intranasal application of livestock*

*Intended User: Vaccination as well as maintenance and repair can be carried out by a veterinarian or instructed / trained farm staff.*

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Subject to amendments in technology and design

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You can download an updated version of the complete spare part list for this product under [www.henkesasswolf.de](http://www.henkesasswolf.de) or kindly request the same.

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© copyright Henke-Sass, Wolf GmbH