HSW ROUX-REVOLVER®

-  Bedienungsanleitung
-  Instruction Manual
-  Manual de instrucciones
-  Mode d'emploi

For veterinary use only

CONTENT

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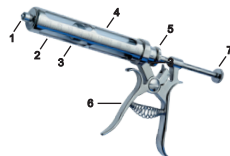
10ml, 30ml, 50ml

ENG

Dosing syringe for multiple dosing of livestock

- 1) Metal needle hub, LL
- 2) Window sleeve
- 3) Clampable piston
- 4) Glass cylinder
- 5) Dosing wheel
- 6) Ergonomic Handle
- 7) Clamping Screw for piston

The HSW ROUX-REVOLVER® is designed for heavy-duty use. The syringe is easy to disassemble and clean. The product must be cleaned and disinfected with appropriate agents after each use. Please follow the instructions provided by the manufacturers of the disinfectants you use.

Inserting the needle:

! Please don't remove the protective cap from the needle (when using disposable needles) until you're ready to use the syringe.

- Attach the needle to the Luer Lock connector and secure it by turning the needle connector clockwise.
- To remove the needle, follow the steps in reverse. Replace the protective cap.

Priming the syringe:

- Squeeze the handle.
- Turn the piston counterclockwise using the locking screw until the piston rod can be retracted with slight resistance and the piston rests against the glass cylinder.
- Pull the piston rod backward and draw up the medication.
- Turn the piston clockwise using the tension screw to increase the preload (depending on the viscosity of the medication).
- The syringe is now ready for use.

Adjusting the dosing volume:

- The desired dosage can be set on the dosing wheel, depending on the model:
 - in 1ml increments for 30ml & 50ml models: 1-2-3-4-5 ml
 - in 0.5ml increments for 30ml model: 0.5-1-1.5-2-2.5 ml
 - in increments 0.25-0.5-1 ml for 10ml model
 - Additional custom increments can also be set as desired

Maintenance after use:

- After application, remove any residue using standard cleaning agents and rinse the syringe thoroughly under warm water.

Disinfection:

! It is strongly recommended to disinfect the product and needles before and after each use.

- Clean the syringe with hot water. Use a bottle filled with hot water for this.
- Place the entire syringe, including the attached needles, in a container of water and boil it for a maximum of 20 minutes.
- After boiling, remove the syringe, dry it with a cleaning cloth, and flush it with water until dry.

Lubrication:

- Unscrew the window sleeve from the handle bracket.
- Lubricate the piston with 100% pure extra virgin olive oil, cold-pressed, or an equivalent edible oil.
- Please follow the manufacturer's instructions regarding shelf life and storage.
- Do not use mineral oils, silicone oils, WD-40, glycerin, or alcohol.
- Petroleum-based oils can damage the O-rings or reduce the product's service life.

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General warnings for all products of the HSW ROUX-REVOLVER® family:

- ! The HSW ROUX-REVOLVER® is only intended to be used for the treatment of animals. The application on humans may cause damage to their health.
- ! Do not point the HSW ROUX-REVOLVER® towards any part of your body or towards other people! This can lead to injuries or accidental injections.
- ! In case of accidental injection or application of a substance, consult a doctor immediately and present the information sheet of the respective substance to the doctor.
- ! The use of non-approved substances or the overdosage / underdosage of these approved substances can lead to adverse health effects in animals and/or later in humans.
- ! Caution, parts of the ROUX-REVOLVER® can break / be damaged. Damaged parts can cause injuries to the operator and animals.
- ! Caution, sharp needles and broken needles can cause harm to the operator and animals. Use needles with care and do not bend them.

General notes for all products of the HSW ROUX-REVOLVER® family:

- Substances to be administered with the HSW ROUX-REVOLVER® are not provided by HSW. The manufacturer's instructions must be adhered.
- The HSW ROUX-REVOLVER® may only be used by operators who have read and understood the user manual in full.
- It is recommended to wear protective goggles, a mouth guard and rubber gloves during the use and cleaning of the HSW ROUX-REVOLVER®.
- Transport and store in dry and ambient conditions.

Explanation of packaging symbols:

eIFU:

- This symbol indicates that a digital instruction manual is available for this this syringe. The digital instruction manual can be found under the following link: www.henkesasswolf.de/manuals

Recycling codes:

- The recycling codes provide information about the material used and can be helpful for waste separation.
- The letter abbreviation PAP identifies cardboard and paper. PAP 20 = Corrugated cardboard

Disposal:

- The HSW ROUX-REVOLVER® products can be disposed of in household waste.
- Due to the risk of injury, cannulas / needles must be disposed of as hazardous waste in accordance with the regulations of each country.

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Intended Use & Intended User:

Intended Use: Intramuscular / subcutaneous vaccination & intranasal application of livestock

Intended User: Vaccination as well as maintenance and repair can be carried out by a veterinarian or instructed / trained farm staff.

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You can download an updated version of the complete spare part list for this product under www.henkesasswolf.de or kindly request the same.

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© copyright Henke-Sass, Wolf GmbH