




**HENKE
SASS
WOLF**

**HSW UNI-MATIC® 5 ml Schlauch / Tube
Ersatzteilliste / Spare part list 02/25**

**Gültig ab / valid
01.03.2025**

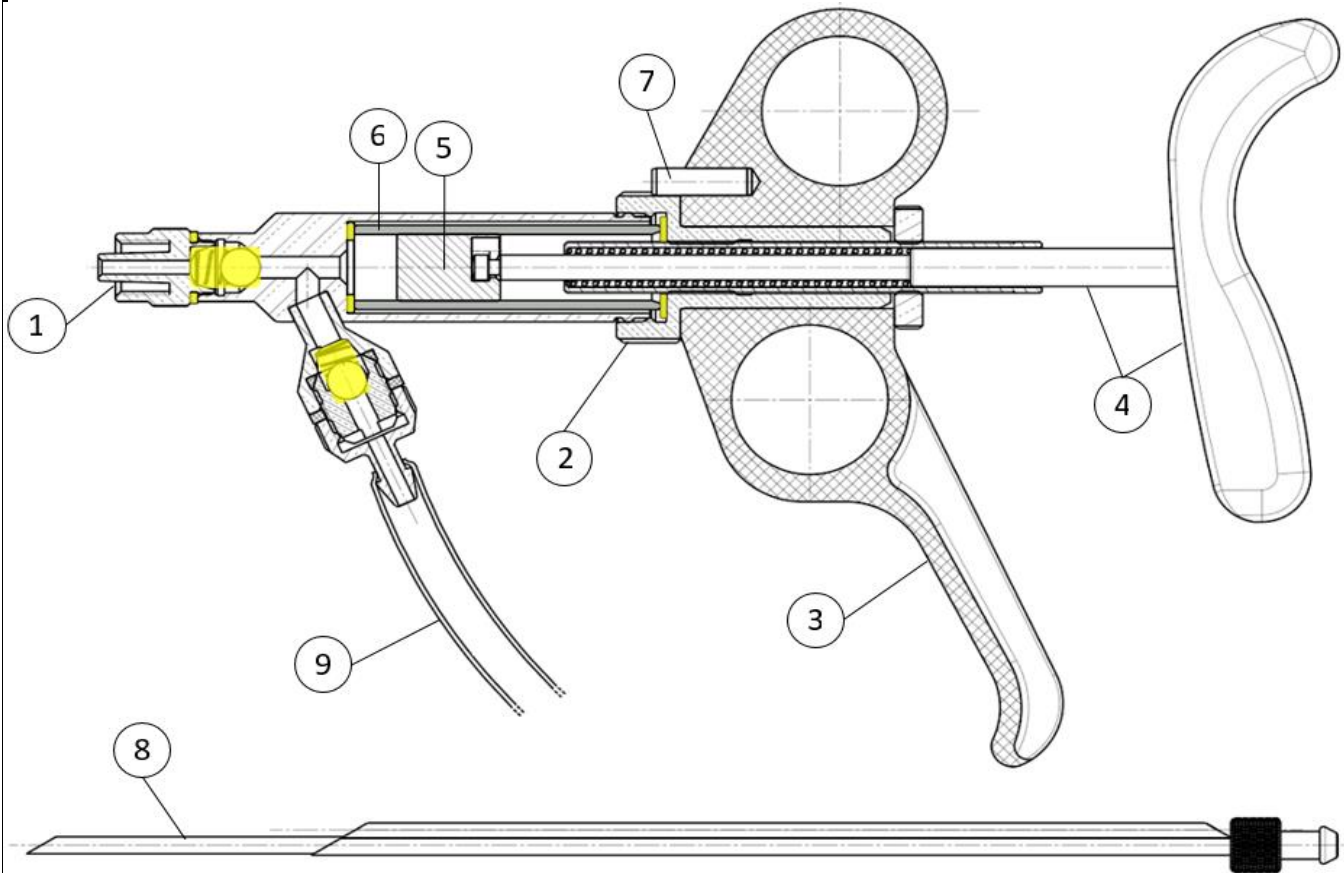

Nr.	Bezeichnung	Description	Appellation	Art. No.	MOQ
1	Vorderteil LL	Nozzle Luer-Lock	Embout Luer-Lock	3000330809	
2	Führungshülse	Sleeve	Douille	8300016522	
3	Griff schwarz	Handle black	Poignée noire	3000000800	5
4	Kolbenstange m. Griff	Piston rod w. handle	Tige de piston complète	8300004031	
5	Kolben	Piston	Piston	8300003175	
6	Glaszylinder 5 ml	Glass barrel 5 cc	Cylindre en verre 5 ml	8300005960	
7	Zylinderstift	Pin	Goujon	3000000812	5
8	Steigrohr 18 cm	Ascending tube 18 cm	Tube à l'ascension	3018760000	
9	Silikonschlauch	Silicone tube	Tuyau silicon	3532000600	

Ersatzteilsets / Spare Part Sets / Set pièces de rechange

-	Ersatzteilset, bestehend aus: 2 x Ventilkugel, 2 x Ventilsfeder, Dichtringen vorne und hinten f. Zylinder, Dichtring für LL Vorderteil	Spare parts set, consisting of: 2 x valve ball, 2 x valve spring, front and rear sealing rings for cylinder, sealing ring for LL front part	Set de pièces de rechange, comprenant : 2 x bille de soupape, 2 x ressort de soupape, bagues d'étanchéité avant et arrière pour cylindre, bague d'étanchéité pour partie avant LL	8300005968	
---	---	---	---	------------	--