HENKE - PRESS

Henke-Sass, Wolf
Abformmasse - Spritze

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WIAS 09/08 740144

HENKE - PRESS

Henke-Sass, Wolf
Application syringe for
Impression materials

HENKE - PRESS **Instructions for use**

Henke-Press syringe consists of:

- syringe cylinder
- syringe piston
- screw plug to attach adapter
- 5 syringe adapters
- cleaning brush

Instructions for use:

With the screw plug the adapter is attached to the syringe.

Filling of the application syringe:

After mixing is complete, filling of the syringe is prepared by spreading out some of the mixed material on approx. half of the mixing pad in a thin layer. With the open end of the syringe cylinder the material is scraped in to the syringe. The piston is inserted - and the syringe is now ready for use.

Cleaning:

Remove adapter after set of the impression material, push out residual elastomer with the piston. Use brush for final cleaning of syringe cylinder. After cleaning the syringe must be sterilized in a commercially available sterilizer before being used again (in an Autoclave at max. 134° C).

The application, use and processing of the products are beyond our control, and therefore, entirely your own responsibility. Should, in spite of this, liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

Manufacturer: Henke-Sass, Wolf GmbH
Keltenstr. 1 • D-78532 Tuttlingen

WIAS 09/08 740144

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Hersteller: Henke-Sass, Wolf GmbH